



The New Mains MiniLoop®

Create stronger relationships with your deaf and partially hearing customers and comply with the Disability Discrimination Act.


Background

The New MiniLoop® is a specially designed and patented induction loop system, shown by customer research to make conversations with hearing aid wearers (whether customers or staff) easier and more relaxed. Combined with staff training, the new MiniLoop® gives a clear demonstration of a strong customer care policy



Main Benefits

- Clearer, more relaxed, less time consuming conversations
- The "Privacy Area" (dotted area) gives greater confidentiality
- Conversations are unaffected by security screens
- Eliminates the embarrassment of having to shout

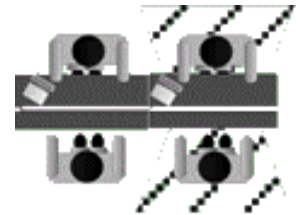

Installation

Installations are carried out by our specially trained staff and normally take approximately 30 minutes per unit. The MiniLoop® box (140mm x 120mm x 25mm) is fitted underneath the counter with a single wire connection to the desktop microphone. Having been plugged in the MiniLoop® is ready for use. All installations are commissioned on-site and demonstrated to staff members.



Ease of Use

Plugged into a 13-amp socket, the system operates without involving staff. Hearing aid wearers seeing the sign will use the "T" switch on their hearing aids.

The MiniLoop®
The MiniLoop®
The MiniLoop®

QUALITY HEARING ASSISTANCE WHEREVER YOU NEED IT

The New Mains MiniLoop®

Technical Specification

DIMENSIONS	Loop Box	140mm x 120mm x 25mm
	Microphone Box	60mm x 80mm x 40mm
POWER SUPPLY	3 Pin Mains Transformer	Regulated industrial fixed voltage
	Input	240 v AC 50Hz 90mA
	Output	15v DC Regulated 1.0A
MICROPHONE	Frequency	50Hz to 12kHz
	Impedance	300 Ohms
	Sensitivity	-60+/- 3dB
	Operating Voltage	1.5v to 10v
	Dimensions	6mm x 5mm
	Signal to Noise Ratio	> 40dB
UNIT	Current Drain	0.4mA Max.
	Output	Up to 7watts at 500mA
	Frequency response	50Hz to 5kHz (+/-3dB around 1kHz)

Organisations using the MiniLoop® include:

- Woolworths
- BT
- Comet
- Local Councils
- The Police
- Sainsburys Supermarkets
- Cheltenham & Gloucester
- Debenhams plc
- Odeon Cinemas
- Abbey National
- B & Q
- Citizens Advice Bureaux
- Health Boards
- Marks & Spencer plc
- Tesco

Staff Training Programmes

Training has been shown by consumer research amongst hard of hearing customers to be as important as the MiniLoop® equipment itself. We provide specially developed staff training together with product familiarisation, through our Hearing Friendly® Training programmes. Training is provided locally on the day of installation and can also be provided to organizations on a regional basis. Programmes for training managers to cascade to staff are also available.

Periods Trial

Monitored test installations can be arranged to develop a clear understanding of the benefits to staff and customers.

iHear Ltd

A wholly owned subsidiary of Adaptive Venture Managers Ltd, a company dedicated to continuously improving the lifestyles of deaf and partially hearing people, by the development of enhancement systems.

Organising a demonstration

For further information or to organise a demonstration or trial, please contact:

Our Product Support Team
on 01506 497 158
or at operations@ihear.co.uk

The MiniLoop®
The MiniLoop®
The MiniLoop®

QUALITY HEARING ASSISTANCE WHEREVER YOU NEED IT